

Celltrion DiaTrust™
COVID-19 Ag
Home Test

1
TEST



EASY Sample
Collection

15
mins

Results in
15 Minutes

Celltrion DiaTrust™
COVID-19 Ag
Home Test

IVD OTC






CELLTRION
USA

Celltrion DiaTrust™
COVID-19 Ag
Home Test



Scan the QR code through your smartphone (Android 10 or newer, iOS 14.2 or newer) camera to download the free Celltrion DiaTrust™ COVID-19 Ag Home Test App (CELLTRION SAFEKEY). Celltrion DiaTrust™ COVID-19 Ag Home Test App can also be accessed through <https://celltrion.safekey.tools> if any error occurs with QR code. **Please Follow the Step-by-Step Instructions Available on the Mobile App.**


If you have symptoms of COVID-19, you can use a single test. If you do not have symptoms of COVID-19, you will need at least two tests per person. You may need to purchase additional tests to perform serial (repeat) testing. This test is more likely to give you a false negative result when you have COVID-19 than a lab-based molecular test.

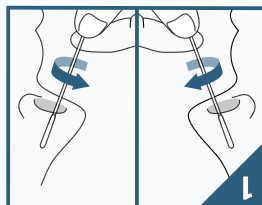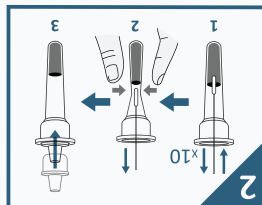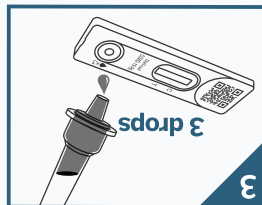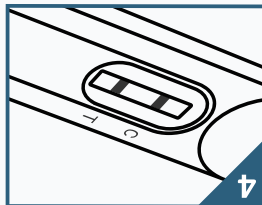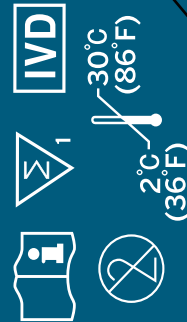

Only open the aluminum pouch when you are ready to do the test.

Please carefully read the precautions outlined in the Instructions for Use manual prior to starting your test. Then please refer to the mobile app and follow the detailed instructions required to collect your sample. Wash or sanitize your hands and dry them thoroughly before starting the test. Make sure they are completely dry. This test involves taking a sample from deep inside your nose. When performing the test, pay particular attention to the instructions on how to swab your nose.

PRECAUTIONS BEFORE THE TEST

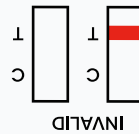


INTERPRETATION OF RESULT

For use under the Emergency Use Authorization (EUA) only. This test is intended to be used as an aid in the diagnosis of a current infection with the virus that causes COVID-19. This test is intended for individuals aged 14 years and older only. **Do not use on children under 14 years of age.**

CONTENTS

- Test Device (1 ea.)
- Test Tube (Extraction Buffer) (1 ea.)
- Filter Cap (1 ea.)
- Swab (1 ea.)
- Instructions for Use (1 ea.)

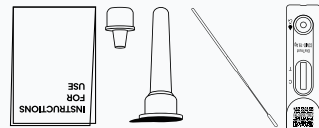

REF CT-P60 D-2 05

Celltrion DiaTrust™
COVID-19 Ag
Home Test

2
TESTS



EASY Sample
Collection

15
mins

Results in
15 Minutes

Celltrion DiaTrust™
COVID-19 Ag
Home Test

IVD OTC



Humasis Co., Ltd.
Site 1: Rm. 114, 502, 504, 604, 604-1,
B03-01, B03-02, 88, Jeonpa-ro,
Dongan-gu, Anyang-si, Gyeonggi-do,
14042, Republic of Korea
Site 2: 37, Gunpocheomdansaneop
2-ro, Gunpo-si, Gyeonggi-do, 15880,
Republic of Korea

CELLTRION Distributed by
Celltrion USA, Inc
One Evertrust Plaza Suite 1207,
Jersey City, New Jersey, 07302, USA
Tel: (201) 499-1844
Email: DiaTrust@celltrion.com
Customer Service: CelltrionUSA.CS@celltrion.com
www.DiaTrustCOVID.com





The Celltrion DiaTrust™ COVID-19 Ag Home Test is for use under Emergency Use Authorization (EUA) only. This product has not been FDA cleared or approved, but has been authorized by FDA under an EUA. This product has been authorized only for the detection of proteins from SARS-CoV-2, not for any other viruses or pathogens. The emergency use of this product is only authorized for the duration of the declaration that circumstances exist justifying the authorization of emergency use of *in vitro* diagnostics for detection and/or diagnosis of COVID-19 under Section 564(b)(1) of the Federal Food, Drug and Cosmetic Act, 21 U.S.C. § 360bbb-3(b)(1), unless the declaration is terminated or authorization is revoked sooner.

Celltrion DiaTrust™
COVID-19 Ag
Home Test



Scan the QR code through your smartphone (Android 10 or newer, iOS 14.2 or newer) camera to download the free Celltrion DiaTrust™ COVID-19 Ag Home Test App (CELLTRION SAFEKEY). Celltrion DiaTrust™ COVID-19 Ag Home Test App can also be accessed through <https://celltrion.safekey.tools> if any error occurs with QR code. **Please Follow the Step-by-Step Instructions Available on the Mobile App.**


If you have symptoms of COVID-19, you can use a single test.
If you do not have symptoms of COVID-19, you will need at least two tests per person. You may need to purchase additional tests to perform serial (repeat) testing.
This test is more likely to give you a false negative result when you have COVID-19 than a lab-based molecular test.

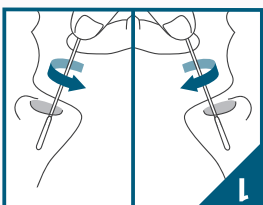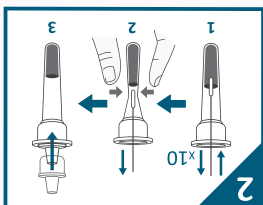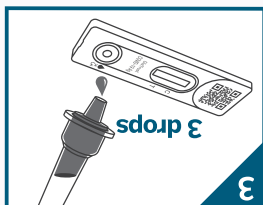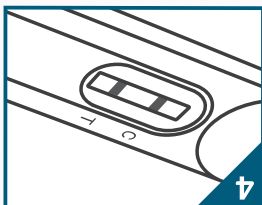


Only open the aluminum pouch when you are ready to do the test.

Please carefully read the precautions outlined in the Instructions for Use manual prior to starting your test. Then please refer to the mobile app and follow the detailed instructions required to collect your sample.
Wash or sanitize your hands and dry them thoroughly before starting the test. Make sure they are completely dry.
This test involves taking a sample from deep inside your nose. When performing the test, pay particular attention to the instructions on how to swab your nose.

PRECAUTIONS BEFORE THE TEST

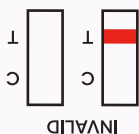

INVALID

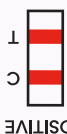


POSITIVE



NEGATIVE

INTERPRETATION OF RESULT

REF CT-P60 D-2 05

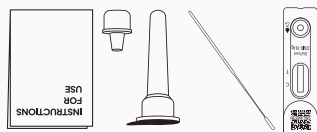

- Test Device (2 ea.)
- Test Tube (Extraction Buffer) (2 ea.)
- Filter Cap (2 ea.)
- Swab (2 ea.)
- Instructions for Use (1 ea.)

CONTENTS

For use under the Emergency Use Authorization (EUA) only.
This test is intended to be used as an aid in the diagnosis of a current infection with the virus that causes COVID-19.
This test is intended for individuals aged 14 years and older only.
Do not use on children under 14 years of age.

Celltrion DiaTrust™
COVID-19 Ag Home Test



21-11-210




For use under the Emergency Use Authorization (EUA) only.
This test is intended to be used as an aid in the diagnosis of a
current infection with the virus that causes COVID-19.
Do not use on children under 14 years of age.



Celltrion DiaTrust™ COVID-19 Ag Home Test

IVD OTC

Only open the aluminum pouch when you are ready to do the test.

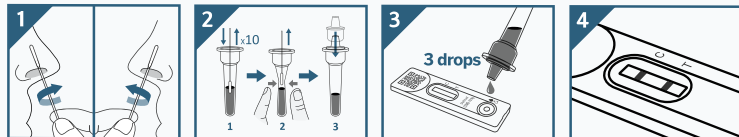

INTERPRETATION OF RESULT



PRECAUTIONS BEFORE THE TEST

Please carefully read the precautions outlined in the Instructions for Use manual prior to starting your test. Then please refer to the mobile app and follow the detailed instructions required to collect your sample.
Wash or sanitize your hands and dry them thoroughly before starting the test. Make sure they are completely dry.
This test involves taking a sample from deep inside your nose. When performing the test, pay particular attention to the instructions on how to swab your nose.



Scan the QR code through your smartphone (Android 10 or newer, iOS 14.2 or newer) camera to download the free Celltrion DiaTrust™ COVID-19 Ag Home Test App (CELLTRION SAFEKEY). Celltrion DiaTrust™ COVID-19 Ag Home Test App can also be accessed through <https://celltrion.safekey.tools> if any error occurs with QR code.
Please Follow the Step-by-Step Instructions Available on the Mobile App.

Rev. 00 / 2021-07-26
TY / PB-228C2HU04-E005



REF CT-P60 D-2 05



If you have symptoms of COVID-19, you can use a single test.
If you do not have symptoms of COVID-19, you will need at least two tests per person. You may need to purchase additional tests to perform serial (repeat) testing.
This test is more likely to give you a false negative result when you have COVID-19 than a lab-based molecular test.

Celltrion DiaTrust™ COVID-19 Ag Home Test


5 TESTS



15 mins Results in 15 Minutes

CONTENTS

- Test Device (5 ea.)
- Test Tube(Extraction Buffer) (5 ea.)
- Filter Cap (5 ea.)
- Swab (5 ea.)
- Instructions for Use (1 ea.)

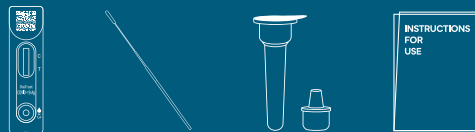

The Celltrion DiaTrust™ COVID-19 Ag Home Test is for use under Emergency Use Authorization (EUA) only. This product has not been FDA cleared or approved, but has been authorized by FDA under an EUA. This product has been authorized only for the detection of proteins from SARS-CoV-2, not for any other viruses or pathogens. The emergency use of this product is only authorized for the duration of the declaration that circumstances exist justifying the authorization of emergency use of *in vitro* diagnostics for detection and/or diagnosis of COVID-19 under Section 564(b)(1) of the Federal Food, Drug and Cosmetic Act, 21 U.S.C. § 360bbb-3(b)(1), unless the declaration is terminated, or authorization is revoked sooner.

Humasis Co., Ltd
Site 1: Rm. 114, 502, 504, 604, 604-1, B03-01, B03-02, 88, Jeonpa-ro, Dongan-gu, Anyang-si, Gyeonggi-do, 14042, Republic of Korea
Site 2: 37, Gunpocheomdansaneop 2-ro, Gunpo-si, Gyeonggi-do, 15880, Republic of Korea

Distributed by Celltrion USA, Inc.
One Evertrust Plaza Suite 1207, Jersey City, New Jersey, 07302, USA
Tel: (201) 499-1844
Email: DiaTrust@celltrion.com
Customer Service: CelltrionUSA.CS@celltrion.com
www.DiaTrustCOVID.com

